







Lab Reference Grid – CT/NG & TV media

EPIC Test Code	Swab/Container	Image	Source	Performing Lab
LAB6059 (CTGCM) Chlamydia trachomatis/ Neisseria gonorrhea	COBAS Dual Swab Kit		Woven Swab= vaginal collections Flocked Swab= endocervical collections, please use woven swab as cleaning swab	TTMC
LAB6059 (CTGCM) Chlamydia trachomatis/ Neisseria gonorrhea	COBAS Urine Kit		First catch “dirty” urine 10-50mL Note: must wait 1 hour if patient recently voided or collected a clean catch sample	TTMC
LABTVMDU Trichomonas only – Male or Female	Sterile UA container		First catch “dirty” urine 20-50mL Note: must wait 1 hour if patient recently voided or collected a clean catch sample	Waconia
LABCTGCT Chlamydia trachomatis/ Neisseria gonorrhoeae by PCR, Throat	Xpert Swab		Throat	Waconia

<p>LABCTGCR</p> <p>Chlamydia trachomatis/ Neisseria gonorrhoeae by PCR, Rectal</p>	<p>Xpert Swab</p>	 <p>The image shows the GeneXpert Xpert® Swab Specimen Collection Kit. It includes a red cap, a yellow swab, and a yellow tube. The packaging is white with blue and red text and logos. The text on the packaging includes 'GeneXpert', 'Xpert® Swab Specimen Collection Kit', 'UK CA', 'CE IVD', '10°C', '2', 'i', 'Cepheid', and '191446 L 231562003 EXP. 2025-10-31'.</p>	<p>Rectal</p> <p>NOTE: Carefully insert the swab approximately 1 cm beyond the anal sphincter (so that the fiber tips are no longer visible) and rotate gently for 10 seconds. Withdraw the swab carefully. Check swab for acceptability (no fecal matter or very little fecal matter)</p>	<p>Waconia</p>
<p>LABMCTGC</p> <p>Chlamydia/ Gonorrhea Amplified RNA</p>	<p>Aptima Collection Multitest Swab</p>	 <p>The image shows the Hologic Aptima Collection Multitest Swab. It includes a pink swab and a white tube with an orange cap. The packaging is white with black text and logos. The text on the packaging includes 'Hologic', 'Aptima', 'Collection Multitest Swab', '10°C', '2', 'i', 'Cepheid', and '191446 L 231562003 EXP. 2025-10-31'.</p>	<p>Eye</p>	<p>Mayo</p>